Rosonia

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Markkinoija
FLOREALIS EHF.
Sidumuli 25
108 Reykjavik – Islanti

Rosonia

For treatment of vulvovaginitis
Vaginal Foam

Adjuvant medical device for the treatment of vulvovaginitis caused by bacterial, mycotic and viral infections, including candidiasis and warts, the treatment of postpartum lesions, and the prevention of herpes simplex 2 (HSV-2) recurrence.

Use

The medical device Rosonia™ is a protective foam which acts as an adjuvant in the treatment of vulvovaginitis caused by bacterial, mycotic and viral infections, including candidiasis and warts, in the treatment of postpartum lesions, and in the prevention of herpes simplex 2 (HSV-2) recurrence. The foam forms a protective barrier on the external genital region, to isolate and protect tissue from external irritating factors. It is a soft foam based on TIAB, a titanium dioxide complex functionalised with monovalent, covalently bonded silver, whose main action is to form a barrier protecting tissue, and create the best condition to prevent relapses and bacterial, mycotic and viral manifestations. Hyaluronic acid significantly supports the filmogenic action of the device, producing a barrier effect. In addition, based on the ancillary action of the *Malva sylvestris* extract, the medical device performs a moisturizing, soothing, demulcent, refreshing and softening function, since it is rich in vitamins, minerals and amino acids, to quickly relieve the typical symptoms of inflammation caused by herpes (burning, tingling), candida and bacteria; furthermore, it improves tissue tone and elasticity, thereby promoting the healing of possible lesions; it also acts as an adjuvant in maintaining physiological immunity.

Instructions for use

Shake before use. Before use, clean the part to treat, and dry it thoroughly. Apply a thin layer of foam on the affected part. Massage until dried. Repeat application once or several times a day. The medical device must be applied on the affected areas upon appearance of the first symptoms (itching, burning, reddening etc.).

Warnings

Do not use the product for any applications other than those stated in the package leaflet. Before any treatment, wash hands thoroughly. Following application, massage the affected area until dried; the foam should form a protective filmogenic layer. Avoid contact with eyes. In case of contact, wash immediately and thoroughly with water. Do not use the device after the expiry date which is stated on the packaging. For topical use; do not swallow. Do not use if the package is open

or damaged. Do not sterilise. Do not use in case of known hypersensitivity to any of the components. Discontinue treatment in case of abnormal skin reactions. In case of prolonged use, consult a doctor. The device is neither a spermicide nor a contraceptive.

Composition

Water, allantoin, glycerin, benzalkonium chloride, sodium hyaluronate, steareth-2, steareth-21, caprylic/capric triglyceride, TIAB (micro-crystals of titanium dioxide with monovalent silver ions covalently linked), polyvinylpyrrolidone, Malva glycolic extract, menthyl lactate.

Contraindications

There are no known phenomena of overdose. Known hypersensitivity to any of the components. There are no known interactions with drugs and medicinal substances.

Keep away from heat sources and naked flames – do not smoke. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Pressure vessel: may explode if heated. Do not pierce or burn, not even after use. Keep out of the reach of children. In case of ingestion or contact with eyes, immediately contact a medical care service or an anti-poison centre, and provide the information on the product stated in the label.

Manufactured by
NMTECH Italia Srl
Via Vittorio Veneto 7
00187 Rome – Italy

Licensed by
LINNEA SA
Via Cantonale
CH 6595 Riazzino – Switzerland

Distributed by
FLOREALIS EHF.
Sidumuli 25
108 Reykjavik – Iceland

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