Rosonia VagiCaps Vaginal softgel capsules

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Intended use

Medical device adjuvant:

- In the treatment of bacterial, fungal and viral infections such as candidiasis and warts
 - In the treatment of post-partum lesions
- Rosonia VagiCaps are softgel capsules that dissolve fast in the vagina. Thanks to the properties of their ingredients, they exert a defensive barrier effect in the adjuvant treatment of bacterial, fungal and viral infections such as vulvovaginitis, candidiasis, genital herpes and warts, and post-partum lesions.
- It forms a protective barrier on the lesion in order to isolate and protect the tissues from external

irritants and to facilitate a normal physiological cicatrisation and the healing process.

The medical device provides a dual action:

- It isolates the application area from external microbial aggressions
- It limits the microbial colonization to the application area and creates ideal conditions to restore the natural physiological conditions of the tissue

Instructions for use

1. Before use clean the genital area thoroughly
 2. Insert the softgel capsule deep into the vagina
 3. Repeat the application daily for seven days
- The medical device has to be used at the onset of first symptoms (itching, burning, discomforts etc.). Can be used at the same time as other topical treatments.

Warnings

- Topical use only - do not swallow
- Discontinue the treatment in case of abnormal skin reactions
- Use the device only in case of intact epithelium
- Do not use the product for purposes other than those indicated in the package leaflet
- Before application, wash your hands thoroughly
- Store at room temperature and away from heat sources
- Do not freeze
- Keep away from the reach of children
- Do not use in case of known hypersensitivity to any of the components
- In case of prolonged use consult with your doctor
- The device is neither a spermicide nor a contraceptive
- Do not use if package is opened or damaged
- Do not use the medical device after the expiry date stated on the package

Contraindications

Known hypersensitivity to the components.

No overdose episodes were observed.

No known interactions with drugs and medicinal substances were observed.

Storage conditions

Keep away from moisture and heat sources.

If exposed to direct sunlight the product could assume a slight yellowish colour, this does not affect its functional characteristics and shelf-life.

Composition

TIAB (titanium dioxide), micro-crystals with silver ions covalently linked), sodium hyaluronate, *Aloe barbadensis* (*Aloe vera*) extract, caprylic/capric triglyceride, PEG-6 stearate (and) glycol stearate (and) PEG-32 stearate, glyceryl stearate

Revision date: 11/2019

 NTC Srl
Via Luigi Razza 3
20124 Milano Italia

Licensed by:
Linnea SA
Via Cantonale CH
6595 Riazzo, Switzerland

Distributed by:
Florealis ehf.
Sidumula 25
108 Reykjavik, Iceland